# ImmuBlot™ Anti-Nuclear Antibody (ANA) Western Blot Immunoassay

IVD

## PRODUCT INSERT

Code 1170 20 Determinations

## INTENDED USE

The ANA Western Blot Immunoassay is used for the detection and identification of RNP, Sm, Ro/SS-A, La/SS-B, Scl-70, Jo-1 and other anti-nuclear autoantibodies.

## SUMMARY AND EXPLANATION

Antinuclear antibodies (ANA), detected by indirect immunofluorescence, aid in the diagnosis of connective tissue disorders including *systemic lupus erythematosus* (SLE), *mixed connective tissue disease* (MCTD), *Sjögren's syndrome* and *scleroderma*<sup>1-5</sup>. ANA occur in about 95% of SLE patients as well as in patients with other connective tissue diseases. ANA may also occur in other disorders such as chronic active hepatitis and primary biliary cirrhosis<sup>6-8</sup>.

ANA are not disease specific. The identification of ANA specificities to various antigens are of help as they may be associated with a particular disease or disease subset. ANA specificities can be determined by gel precipitation, hemagglutination, ELISA, or by western (immuno) blot.

Antibodies to U-1 RNP are characteristic of MCTD patients. Antibodies to Sm antigen occur exclusively in SLE. SS-A (Ro) antibodies are detected in *Sjögren's syndrome* and SLE<sup>1</sup>. Antibodies to Jo-1, or histidyl-tRNA synthetase, occur in patients with myositis<sup>1,3</sup>. SS-B (La) is primarily reported in SLE and *Sjögren's syndrome*. Antibodies to both the 52 kD SS-A (Ro) and the 48 kD SS-B (La) are found in virtually all patients with *neonatal lupus erythematosus* and *complete heart block*, and the presence of both in SLE indicates a later age of disease onset and a lower frequency of *lupus nephritis*<sup>9</sup>. Scl-70 antibodies react with *human topoisomerase* and are found in patients with *diffuse scleroderma*.

## PRINCIPLES OF PROCEDURE

In the ImmuBlot™ ANA Western Blot Immunoassay, the nuclear and cytoplasmic antigens from a HeLa cell line, supplemented with additional antigens, are separated according to their molecular weight (MW) by *polyacrylamide* gel electrophoresis and then transferred onto a PVDF membrane. A set of 3 different MW markers are processed simultaneously in order to facilitate MW identification and hence, interpretation of the results. The additional MW markers (125 kD, 77 kD, 70 kD) appear as blue-violet bands during the staining process. They serve as internal control markers for the detection system.

Antigen containing strips are incubated with control or patient serum. If present in the serum, anti-nuclear antibodies bind to the specific antigen on the strip. After repeated washing and incubations with two types of conjugates and a chromogen substrate, positive reactions are indicated by a blue-violet banding on the strip. The specificity of the antibody is defined by the identification of the positive bands in comparison with the positive control strip, when aligned with the 3 MW markers (125kD, 77kD, 70kD).

For technical assistance please contact:

**IMMCO Diagnostics, Inc.**  
60 Pineview Drive  
Buffalo, NY 14228-2120  
Telephone: (716) 691-0091  
Fax: (716) 691-0466  
Toll Free USA/Canada: 1-800-537-TEST  
E-Mail: [info@immcodiagnostics.com](mailto:info@immcodiagnostics.com)

or your local product distributor

EU Authorized Representative/Autorisierter Repräsentant/Rappresentante  
Autorizzato/Representante Autorizado/Représentant Autorisé  
EMERGO Group, Inc.  
Molenstraat 15, 2513 BH, The Hague,  
The Netherlands  
Tel (+31) 345 8570, Fax (+31) 346 7299  
[www.emergogroup.com](http://www.emergogroup.com)

## REAGENTS

### Storage and Preparation

Store all reagents at 2-8°C. **Do not freeze.**

Do not use if reagent is not clear or if a precipitate is present. All reagents must be brought to room temperature (22-30°C) prior to use.

Coated antigen strips are for one time use only.

### Precautions

All human derived components used have been tested for HBsAg, HCV, HIV-1 and 2 and HTLV-I and found negative by FDA required tests. However human blood derivatives and patient specimens should be considered potentially infectious. Follow good laboratory practices in storing, dispensing and disposing of these materials<sup>10</sup>.

WARNING - Sodium azide (NaN<sub>3</sub>) may react with lead and copper plumbing to form highly explosive metal azides. Upon disposal of liquids, flush with large volumes of water to prevent azide buildup. Sodium azide may be toxic if ingested. If ingested, report incident immediately to laboratory director or poison control center.

Instructions should be followed exactly as they appear in this kit insert to ensure valid results. Do not interchange kit components with those from other sources other than the same catalog number from IMMCO Diagnostics. Follow good laboratory practices to minimize microbial and cross contamination of reagents when handling. Do not use beyond expiration date on the label.

### Materials Provided

ImmuBlot™ ANA Western Blot Immunoassay

Code: 1170

### Kit contains sufficient reagents to perform 20 determinations.

1 x 20	Western Blot Strips, numbered individually at the top.
1 x 120 µl	*Positive control ( <b>red vial cap</b> ), contains antibodies to RNP, Sm, SS-A (Ro), SS-B (La), Scl-70, Jo-1
1 x 120 µl	*Negative Control ( <b>yellow vial cap</b> )
1 x 250 µl	*Conjugate A ( <b>blue vial cap</b> )
1 x 250 µl	*Conjugate B ( <b>white vial cap</b> )
1 x 60 ml	*Blocking Diluent. Ready to use.
1 x vial	Powder Wash Buffer. Reconstitute to one liter
1 x 25 ml	*Enzyme Substrate ( <b>amber bottle</b> ) contains 5-bromo-4 chloro-3 indoiyl-phosphate (BCIP) and nitro blue tetrazolium (NBT)
1	Lot specific Positive Control template to assist in interpretation
3	Assay trays


\*Contains <0.1% NaN<sub>3</sub>

### Materials Required But Not Provided

- Clean 1000 ml graduated cylinder
- Blunt forceps (Filter forceps)
- Rocker or rotating shaker
- Disposable polypropylene tubes for mixing reagents
- Absorbent paper or paper towels
- Deionized or distilled water
- Squeeze bottle to hold diluted wash buffer
- Pipettes capable of delivering 5 µl to 1000 µl
- Disposable pipet tips
- Timer

## NOTES

**Figure 1: Negative Control Strip**

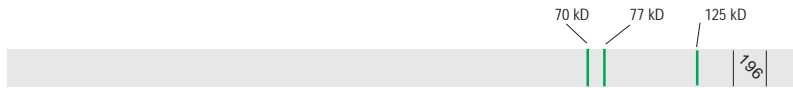The presence of blue-violet band(s) (other than molecular markers) indicate a positive reaction. The lot specific template of the Positive Control which is included in each kit should be used as an interpretation aid only.

**Figure 2: Positive Control Strip**

**Table 1: Prevalence of each Type of ANA in Disease**

Antigen	Disease Specificity	Prevalence	Molecular Weight (primary band in bold)
Sm	SLE	75-90%	<b>B' 29 kD, B 28 kD,</b> and D 16 kD
U-1 RNP	SLE MCTD	35-45% 95-98% (high titer)	<b>68 kD,</b> 34 kD, 22 kD
SS-A/Ro	SLE Sjögrens syndrome	25-35% 50-60%	<b>60 kD,</b> 52 kD
SS-B/La	SLE Sjögrens syndrome	10-21% 40-50%	<b>48 kD,</b> 43 kD
Scl 70	Systemic Sclerosis	20-25%	<b>100 kD</b>
Jo-1	Myositis	25-30%	<b>56 kD</b>

**SPECIMEN COLLECTION AND HANDLING**

Freshly drawn and properly refrigerated sera or plasma should be used in this assay. Do not use icteric, hemolyzed, grossly lipemic or specimens with obvious microbial contamination. Samples can be stored at 2-8°C for 3-5 days or at -20°C or lower for longer periods. Avoid repeated freezing and thawing of samples.

**PROCEDURE**

**Procedural Notes**

- Read Product Insert carefully before starting with the assay.
- Let serum specimens and test reagents equilibrate 30 minutes at room temperature before starting with the test procedure. Return all unused specimens and reagents to refrigerator immediately after use.
- Positive and Negative Control strips must be included in each assay run.
- Proper washing technique is critical to the satisfactory performance of the assay.

**Test Method**

- Step 1.** Using blunt forceps, place required **Strips** with number side up into individual wells of the assay tray.
- Step 2.** Pipet **1.0 ml** of Blocking Diluent followed by **10 µl** of Positive and Negative Control and patient sample into appropriate wells to obtain a **1:101 dilution**. Incubate **60 minutes** (± 5 min) at room temperature on a rocker or rotating shaker.
- Step 3.** Aspirate sample solution into waste container. Thoroughly wash strips with Wash Buffer by squirting solution directly onto strips. Soak strips with gentle agitation for **5 minutes** and aspirate solution into waste container. **Repeat 2x for a total of 3 washes.**

*Caution: Complete washing of the strips between incubations is crucial to obtain valid results. Improper washing will result in high background staining. Position the tip of the squirt bottle approximately half a centimeter above each strip and spray directly down its entire length. Before discarding, rock the tray back and forth, loosening the strip from the bottom of the well and rinsing away any reagent trapped beneath it.*

- Step 4.** Pipet **1.0 ml** of Blocking Diluent followed by **10 µl** of Conjugate A into each well. Incubate **30 minutes** (± 5 min) at room temperature on rocker or rotating shaker.
- Step 5.** Repeat **Step 3.**
- Step 6.** Pipet **1.0 ml** of Blocking Diluent followed by **10 µl** of Conjugate B into each well. Incubate **30 minutes** (± 5 min) at room temperature on rocker or rotating shaker.
- Step 7.** Repeat **Step 3.**
- Step 8.** Pipet **1.0 ml** Substrate into each well. Incubate with gentle shaking **10 minutes** (± 5 min) at room temperature and reduced light.
- Step 9.** Stop development with deionized water (2 x 1 minute washes).
- Step 10.** Using blunt forceps, remove strips from assay tray and place them gently onto absorbent paper. Handle strips only at the ends and let them dry **15-20 minutes**. Mount strips carefully onto sheet of paper by aligning MW markers and store in the dark.

**Quality Control**

If the following specifications are not met, the run is to be considered invalid, and should be repeated.

1. The Negative Control shows no additional bands other than molecular weight markers at 125 kD, 77 kD, and 70 kD. These markers appear on every strip and help in the interpretation of the reacted strips (Figure 1).

- The Positive Control should be positive for Scl-70 (100 kD), SS-A(Ro) (60 kD), Jo-1(56 kD), SS-B(La) (48 kD), RNP (68 kD) and Sm B/B<sup>1</sup> (28,29 kD) as shown in Figure 2. Additional bands may also be present as indicated in Table 1 on the next page.

## RESULTS

The ANA western blot strips contain three major bands at 125kD, 77kD and 70kD molecular weights (refer to figure 1). These bands serve as molecular weight alignment markers. Use the 125kD band and/or the 77kD and 70kD bands to align patient strips with the Control Card.

The presence of blue-violet band(s) (other than molecular markers) indicate a positive reaction. The lot specific template of the Positive Control which is included in each kit should be used as an interpretation aid only.

### Identification of Individual Autoantibodies

A positive test results in the presence of a band that aligns directly to a reference band on the positive control. For example, anti-Sm antibodies primarily yield a doublet at 28 kD and 29 kD. If the sample has a doublet at 28 and 29 kD that aligns with the positive control Sm bands, then the Sm specificity can be assigned to the patient's serum.

Samples may have other bands than specificities shown. Since the antigens contained on the strips are prepared from whole cell extracts, bands of unknown specificities may also be identified.

Samples can demonstrate more than one specificity and may have bands that do not align with bands in controls. Samples that are strongly positive for Sm, have many bands from 140 kD down to 12.5 kD with the B/B<sup>1</sup> bands at 28 kD, 29 kD being the strongest. These additional bands should not be confused with other specificities.

### LIMITATIONS OF THE PROCEDURE

The ImmuBlot™ ANA Kit should be used as an aid to diagnosis. Positive results may be found in apparently healthy individuals. Therefore, results should be interpreted by a medical authority in light of the patient's complete clinical condition.

### EXPECTED VALUES

Expected values in a normal population are negative. The prevalence of frequently detected autoantibodies related to Collagen-Vascular disorders are summarized below.

### TROUBLESHOOTING GUIDE

- Extra bands on Negative Control strip.** Contaminated Negative Control serum, or cross contamination from another well.
- Positive Control appears like Negative Control strip.** Negative Control serum was confused with Positive Control serum.
- Strips are completely blank.** Addition of Conjugate 1 or 2, Substrate and/or specimens may have been omitted.
- High background, poor contrast between bands and background.** Wash step(s) may have been omitted or incorrectly performed, or incubations were overextended.

## REFERENCES•REFERENCIAS•LITERATUR•RIFERIMENTI

- Kumar V, Beutner EH and Chorzelski TP. Autoimmunity of the skin. In "Concepts in Immunopathology", Vol 1, Cruse JM and Lewis RE Jr, Eds, Karger, Basel. 1985, 318-53.
- Reimer G, Cornell RC and Tan EM. The biochemical nature of nuclear antigens reactive with antinuclear antibodies. In "Immunopathology of the Skin", Beutner EH, Chorzelski TP, Kumar V, Eds, John Wiley and Sons, New York, 3rd Ed. 1987, 519-531.
- Tan EM, Chan EKL, Sullivan KF and Rubin RL. Antinuclear antibodies (ANAs): Diagnostically specific immune markers and clues toward the understanding of systemic autoimmunity. Clin Immunol Immunopathol; 1988, 47:121-141.
- von Muhlen CA, Tan EM. Autoantibodies in the diagnosis of systemic rheumatic diseases. Semin Arthritis Rheum; 1985, 21:323-58.
- Tan, EM. Antinuclear Antibodies: Diagnostic Markers for Autoimmune Diseases and Probes for Cell Biology. "Advances in Immunology" Vol. 44; 1987, pp 93-151.
- Manns M, Gerken G, Kyriatsoulis A and Meyer zum Büschenfeld KH. Significant autoimmune markers of autoimmune liver disorders: Current status. J Clin Lab Anal; 1987, 1:362-370.
- Mackay IR. Autoimmunity and the liver. Clin Aspects Immunity. 2: 8- 17, 1988.
- McMillan SA, Alderdice JM, McKee CM et al. Diversity of autoantibodies in patients with anti-mitochondrial antibody and their diagnostic value. J Clin Path 4: 232-236, 1987.
- Ben-Chetrit, E, Fox RI, and Tan EM. Dissociation of Immune Responses to the SS-A(Ro) 52kD and 60kD polypeptides in Systemic Lupus Erythematosus and Sjögren's Syndrome. Arth. Rheum. 33, 349-355, 1990.
- Biosafety in Microbiological and Biomedical Laboratories. Centers for Disease Control, National Institutes of Health, (HHS Pub No [CDC] 93-8395), 1993.

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